



COMMERCIAL RESPIRATOR 95%

INTENDED USE

The Commercial Respirator 95% is intended to be worn to protect both the patient and healthcare personnel from the transfer of microorganisms, body fluids and particulate material. These face masks are intended for use in infection control practices to reduce the potential exposure to blood and bodily fluids. This is a single use, disposable device.



PRODUCT INFORMATION

SPECIFICATIONS:

- Tested According to 42CFR84 Standard*
- Filters 95% or more of relevant particles
- PFE, Particulate Efficiency >95% at 0.0725 microns
- BFE, Bacteria Live Particle Filtration Efficiency >95% 0.1 micron
- Flame Spread Class 1
- ASTM F2100-11 Standard ASTM Level 1

MATERIALS: 4 plies of fabric consisting of:

- 2 layers spun bonded polypropylene
- 2 layers melt blown nonwoven polypropylene
- Nose piece wire sonic welded inside the mask
- Elastic straps to be placed around the head

INSTRUCTIONS FOR USE

1. Open package and remove mask. When wearing the mask, ensure the white side is facing inwards, and metal nose clip is facing up.
2. Place mask against face and wrap loops around ears. Gently pinch the metal nose clip against the bridge of the nose to secure in place.

NOTE: This product is for one-time use. Dispose of it according to the requirements of medical waste management.

*Tested by multiple US Accredited Laboratories - results not evaluated by NIOSH

website | www.rbsigma.com
email | info@rbsigma.com

phone | 440.290.0577
fax | 440.226.8337

6111 Heisley Road
Mentor, OH 44060